


SUTURE PASS LASSO

Suture passers in arthroscopic techniques

- SuturePass is a single use sterile device for suture passage through soft tissue for arthroscopic techniques. It is available with different angles for Bankart, SLAP and rotator cuff arthroscopic repairs.
- Suture Pass has a 2.0 mm external diameter endpoint; it is preloaded with a Nitinol monofilament loop used to carry the suture through soft tissue.
- The sharp point and the small diameter axis penetrate smoothly into the soft tissue; the reinforced axis withstands bending.
- Once the tip has penetrated into the soft tissue, the handle unfolds and it is retrieved through the cannula with a Crochet Hook or Suture Retriever.
- The suture thread or suture tail previously fitted to the loop is placed and pulled from the opposite end of the loop, thus releasing the suture through the tissue and out of the cannula.
- Easy to use.
- Suture Pass will simplify suture manipulation in arthroscopic procedures.

Order Informations

Ref Code	SL-4045L - Left Suture lasso 45°
Ref Code	4068 - STRAIGHT Suture Pass
Ref Code	SL-4045R- Right Suture Lasso 45°

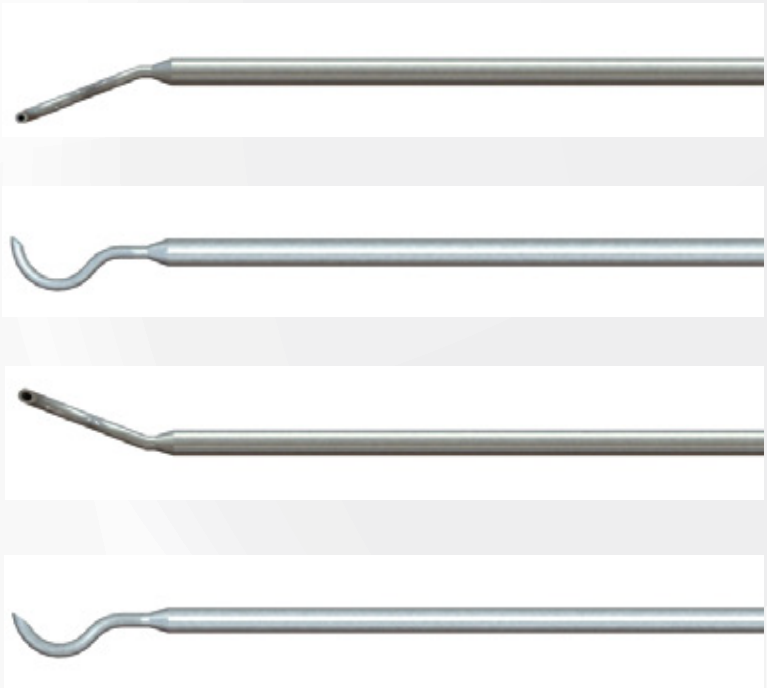

Features

Disposable
Preloaded with nitinol loop
ETO Sterile
Endpoint external diameter of Ø2.0mm

NeoSys

www.neosysmed.com